



Welcome to OnCall 2.0


Table of Contents

GETTING STARTED	<i>Error! Bookmark not defined.</i>
REGISTER	2
ADD DEVICE	4
OnCall 2.0 NEW FEATURES	5
NICKNAMES & SMS ALERTS	5
ALERTS	6
ACTIVELY ALARMING	7
VIEW OPTIONS	8
TECHNICAL SUPPORT	9

Welcome to OnCall.Spectros.com

OnCall is a private and HIPAA compliant remote access web application so you can “see” what your flaps are reading, when you can’t be bedside. You can log-in from any location that has internet access.

REGISTER

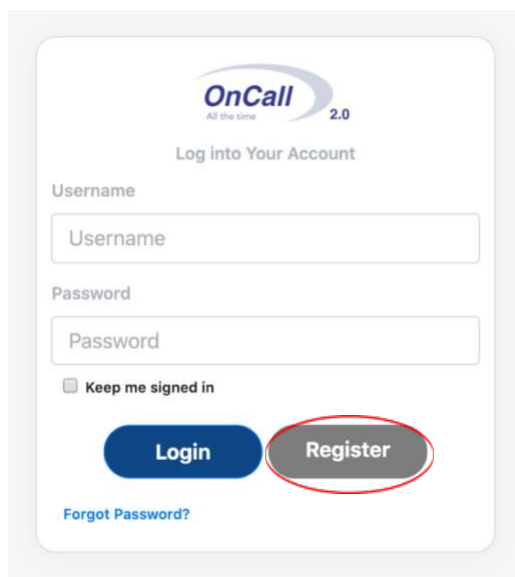A screenshot of the OnCall 2.0 login and registration interface. At the top, the OnCall 2.0 logo is displayed with the tagline "All the time". Below the logo, the text "Log into Your Account" is centered. The form contains two input fields: "Username" and "Password". Below the password field is a checkbox labeled "Keep me signed in". At the bottom of the form are two buttons: a blue "Login" button and a grey "Register" button. The "Register" button is circled in red. A link for "Forgot Password?" is located at the bottom left of the form.

New users need to follow these Registration Steps:

Click the “Register” button located on the bottom right, as shown in capture on the right.

Complete the fields in the Contact Information screen and click the “Submit” button.

OnCall 2.0 Instructional

Contact information

First name*	Last name*	
<input type="text" value="First name"/>	<input type="text" value="Lastname"/>	
Address		
<input type="text" value="Address"/>		
State	City	
<input type="text" value="Select"/>	<input type="text" value="Select"/>	
Country Code	Phone	Extension
<input type="text" value="+1"/>	<input type="text" value="Phone"/>	<input type="text" value="Extension"/>

User information

Username*	Email address*
<input type="text" value="Username"/>	<input type="text" value="Email address"/>


Submit

Cancel

This will trigger a process that will send a temporary password to the registered e-mail account.

Once you receive your login credentials, enter your username and password and click the “Login” button. Once logged in you can change your password by clicking on the top right user icon. A drop-down menu will appear once you click on your user icon and then select “Reset Password.”

The screenshot shows the OnCall 2.0 interface. At the top, there is a navigation bar with a monitor icon on the left, the 'OnCall All the time 2.0' logo in the center, and a user profile icon labeled 'apple' on the right. Below the navigation bar, there are radio buttons for 'All', 'Active', 'Inactive <7 days', and 'Inactive >7 days'. To the right of these buttons is an 'Add device' button. A dropdown menu is open from the user profile icon, showing options: 'Admin Interfaces', 'Reset Password' (circled in red), and 'Logout'. Below the dropdown, there is a table with columns: 'S/N', 'Nickname', 'Alarm', 'Sat%', 'Hb', 'Last', and 'Reset'.

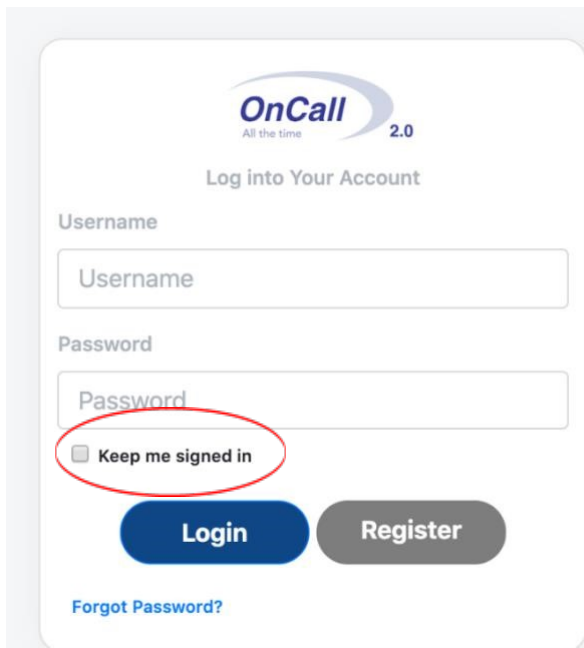


The image shows the OnCall 2.0 login interface. At the top is the OnCall logo with the tagline "All the time" and the version number "2.0". Below the logo is the text "Log into Your Account". There are two input fields: "Username" and "Password". Below the password field is a checkbox labeled "Keep me signed in", which is circled in red. At the bottom of the form are two buttons: "Login" (blue) and "Register" (grey). A link for "Forgot Password?" is located below the "Login" button.

You can stay logged in by checking the box “Keep me signed in” The system will allow you to stay logged in for as long as there is activity within 14 days.

ADD DEVICE

This feature works just as it does on the previous OnCall site.


Click “Add Device” button:

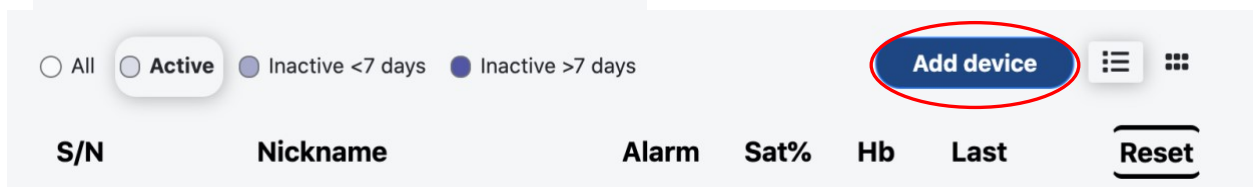

The image shows a user management interface. At the top, there are radio buttons for "All", "Active", "Inactive <7 days", and "Inactive >7 days". The "Active" button is selected. To the right is a blue button labeled "Add device", which is circled in red. Further right are two menu icons. Below this is a table header with columns: "S/N", "Nickname", "Alarm", "Sat%", "Hb", "Last", and "Reset".

Enter the Serial Number & The Pair Key (sample in red) in the indicated fields.

Click “Save Change” button.

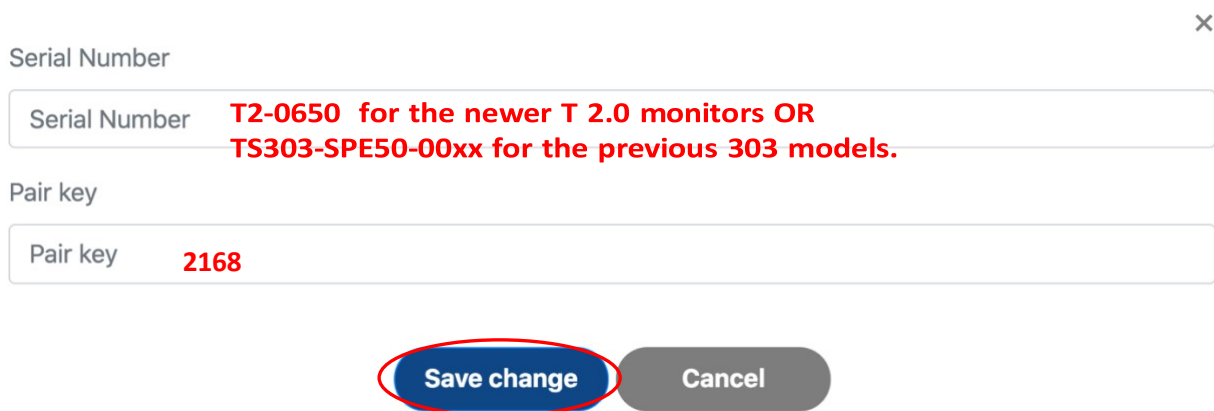


The image shows a device configuration form. At the top right is a close button (X). There are two input fields: "Serial Number" and "Pair key". The "Serial Number" field contains the text "T2-0650 for the newer T 2.0 monitors OR TS303-SPE50-00xx for the previous 303 models." in red. The "Pair key" field contains the text "2168" in red. At the bottom are two buttons: "Save change" (blue) and "Cancel" (grey). The "Save change" button is circled in red.

OnCall 2.0 NEW FEATURES


Devices on each login can be sorted in many ways.

Users can view by All, Active, Inactive for > 7 Days, Inactive for < 7 days by selecting the buttons indicated for each.

The default sort is to bring the newest Nicknamed device to the top, so users can view their cases most recent to oldest. The Reset button can also be used to return you to the default sort.

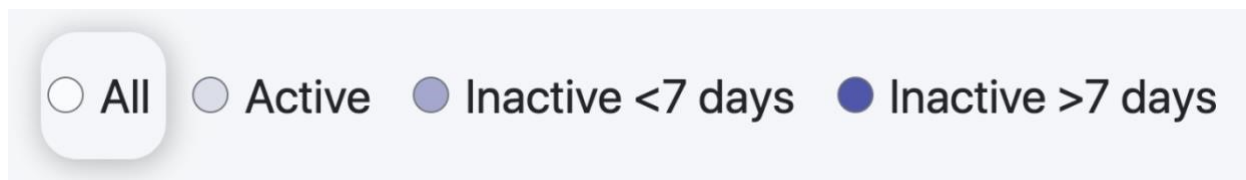

Users can also sort by any of the next menu options found below the Active/Inactive menu bar by clicking in title; ie S/N; Nickname, etc.

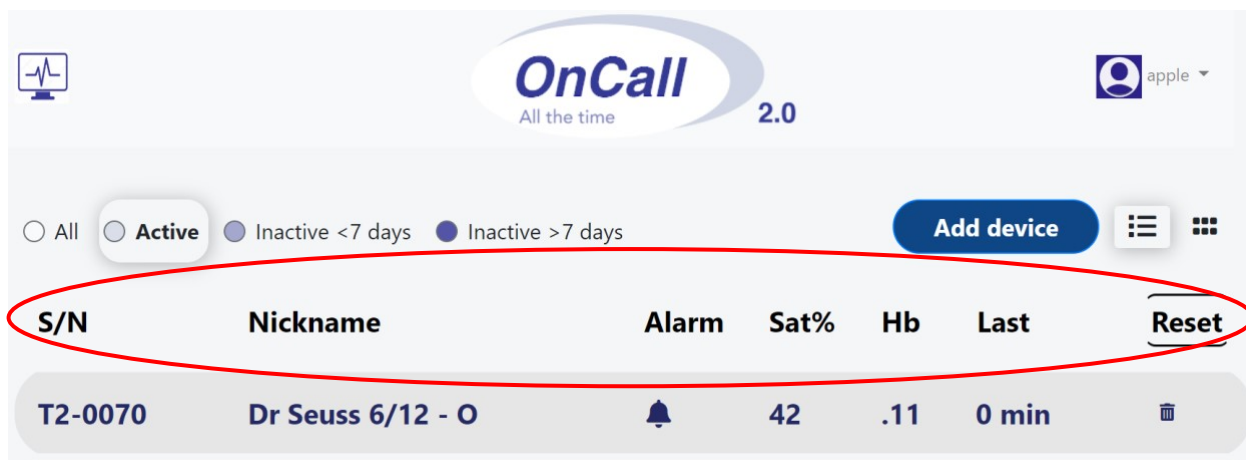

NICKNAMES & SMS ALERTS

The nickname field allows users to add identifying information to discern one flap from another. Most surgeons use their name and/or facility name along with the DOS here. This field allows for free text input. **We strongly recommend no PHI be entered to identify your patients.**

[Add a nickname](#)

SETON

Nickname

SMS Phone #

Stop msgs after x days?

Alarm Range Saturation

Sx Site

Left Right Other

Save change

Cancel

ALERTS

We have added SMS alerts to this field. Should you like to be alerted if T-Stat goes into alarm mode, simply enter a valid mobile phone number in this field and select a time range in # of days for which this mobile number should be notified. The system default is set to stop notifying this user after 5 days. This is in place to help prevent alerts going to a colleague should the device be utilized on a new patient after 5 days and the next user not use this function.

SMS Phone #

Stop msgs after x days?

- ✓ 1
- 2
- 3
- 4
- 5
- 6
- 7

Alarm Range Saturation



Sx Site

Left Right Other

Next – Set the **Low** Alarm limit at which you would like to be notified


**** IMPORTANT**** **This setting is completely independent of the Alarm Limit on the T-Stat.** OnCall 2.0 will NOT alert when the Alarm goes off at the bedside, but **only** when the StO2 level on the website falls below the level selected in this screen. This allows users to be alerted regardless of whether or not bedside staff has properly set alarm limits and/or change the setting to be alerted in the day(s) following surgery.

Alerts may also be accessed by selecting the alarm icon specific to the serial number.





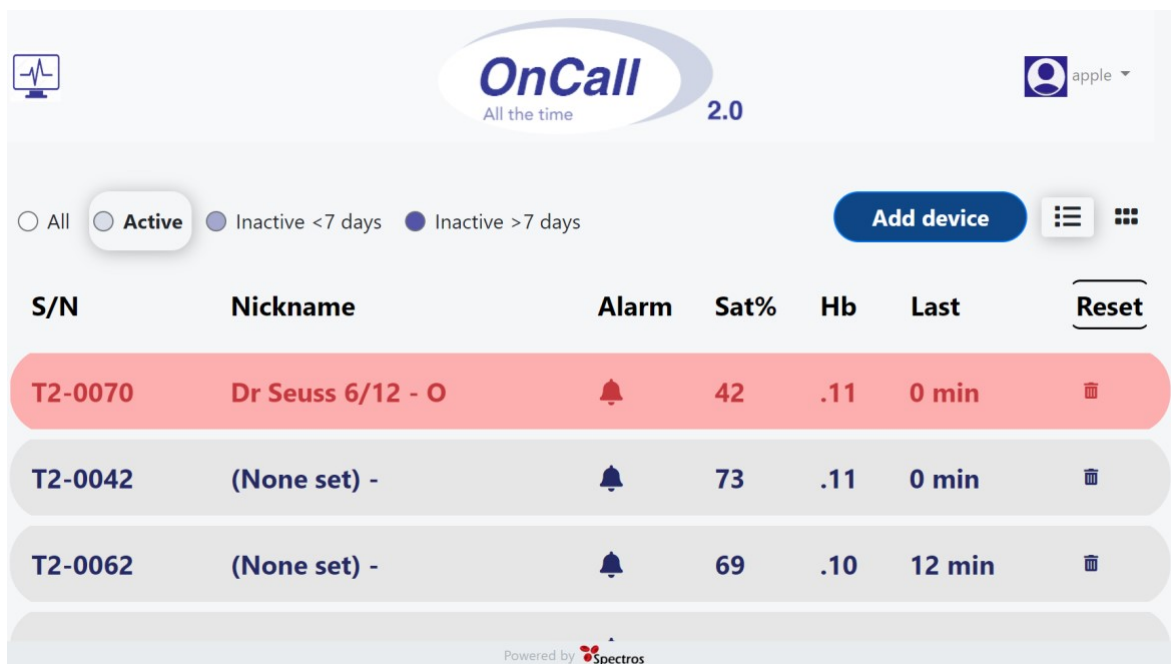

S/N	Nickname	Alarm	Sat%	Hb	Last	Reset
T2-0037	SETON -		64	.07	4512 min	


ACTIVELY ALARMING

Any device in Alarm mode as set here in OnCall 2.0 will be highlighted in red in both list view and graph view as shown below.



The screenshot shows the OnCall 2.0 interface with a list of devices. The first device, T2-0070, is highlighted in red. A red arrow points to this device from the right side of the screen.

S/N	Nickname	Alarm	Sat%	Hb	Last	Reset
T2-0070	Dr Seuss 6/12 - O		42	.11	0 min	
T2-0042	(None set) -		73	.11	0 min	
T2-0062	(None set) -		69	.10	12 min	

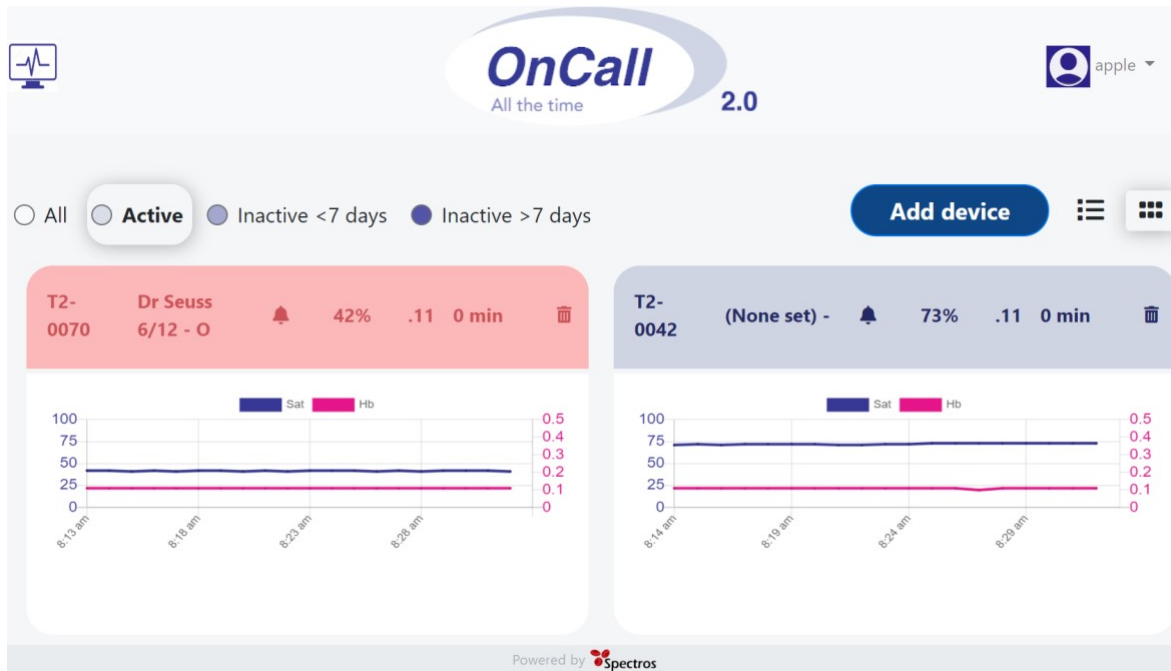

A surgical site field has been added to help the user identify Left form Right in bilateral cases or select Other for other unilateral surgeries.


VIEW OPTIONS

OnCall 2.0 allows users to view their T-Stat devices in either List View or Graph (thumbnail) View – found to the right of the menu items.

The screenshot shows the List View of T-Stat devices. On the left, there is a list view icon (three horizontal lines). On the right, there is a table of device data:

T2-0085	(None set)	63.2	0.10	46 min	Remove
T2-0016	Nickname_02	61.3	0.07	111 min	Remove
T2-0025	Nickname_01	71	0.09	4206 min	Remove

(List View)

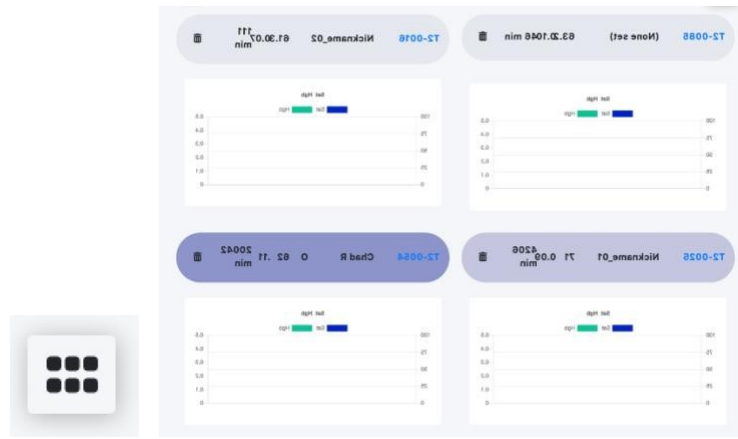

(Thumbnail View)

Both are located on the right of the Add Device button on your main viewing screen.



TECHNICAL SUPPORT

WWW.SPECTROS.COM <https://www.aaihealthcare.com/t-stat>

INFO@SPECTROS.COM

Call: FIND X TSTAT (346-398-7828)

SPECTROS Office (650) 851-4040